NEW USER GUIDE

CAUTION: Improper application, overstretching, or applying on areas of sensitive skin, and improper or quick removal may result in skin irritation, blisters or skin abrasion. Do not place KT TAPE® directly over cuts or open wounds. One or more of the components of this product is a dyed material. There is a possibility that some color migration may occur during use. Instructions provided are for educational use only. KT TAPE® is not a replacement for professional medical care. Warranties and remedies limited to product replacement cost.

TIPS FOR BEST RESULTS

APPLY BEFORE ACTIVITY
Apply tape one hour before beginning activity for best adhesion.

TRIM HAIR
Trim excessive hair for better adhesion.

CLEAN SKIN
Clean dirt, oils and lotions from area with rubbing alcohol.

DON'T RIP OFF!
Gently remove tape from skin


DON'T OVERSTRETCH
Overstretching tape may result in skin irritation and/or abrasions.

END ON SKIN, NOT ON TAPE
Apply directly to skin when possible, especially the ends.


STRETCHING GUIDE AND INDICATOR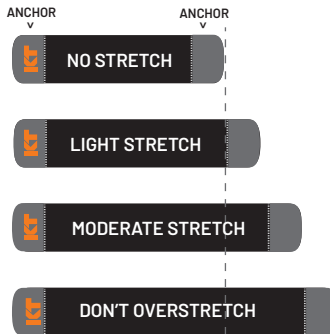

CAUTION:

- For external use only
- Avoid contact with eyes
- Do not apply to wounds or damaged skin
- Do not bandage tightly
- Keep out of reach of children
- If condition worsens, or if symptoms persist for more than 7 days or clear up and occur again within a few days, discontinue use of this product and consult a physician

2 HOW TO APPLY TAPE - END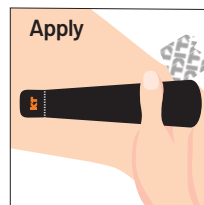


Apply as shown.

- ANCHOR first 5 cm without stretch.
- Apply stretch according to application
- ANCHOR last 5 cm without stretch.
- After applying, rub tape for best adhesion.

Apply as needed. Can be worn up to 7 days.

HOW TO APPLY TAPE - MIDDLE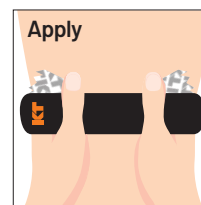


Apply as shown.


- Apply stretch according to application
- ANCHOR first 5 cm without stretch.
- ANCHOR last 5 cm without stretch.
- After applying, rub tape for best adhesion.

Read all cautions on box.

HOW TO REMOVE PAPER BACKING - MIDDLE

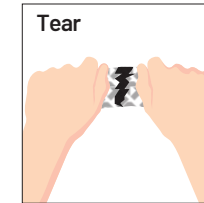
Twist and pull to tear paper in the center.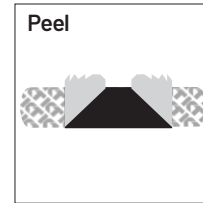

Peel paper leaving 5 cm of paper on both sides for handling.

3 HOW TO REMOVE TAPE

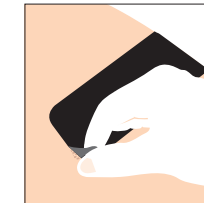Begin by peeling tape up at a corner. Hold skin down where tape is being pulled off with one hand while gently pulling tape off with the other hand. Remove slowly.

See instructions for strong adhesive tape removal on our website

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