REF
MD195

INTENDED USE

211

REUSABLE HOT-COLD GEL PACK

INTENDED USE

Best for use as a hot or cold compress for various conditions such as injuries, muscle pain, and inflammation.

SIZE

One Size

WARNINGS & PRECAUTIONS

This product is not a substitute for medical care. Always seek professional medical advice for the diagnosis and treatment of pain, injury, or irritation. Gel packs are designed to help manage symptoms such as pain and swelling. Use as directed and do not exceed the recommended usage duration to avoid potential skin irritation or injury.

DIRECTIONS

FREEZER:

- Place the gel pack in the freezer for at least 1.5 to 2 hours.
- Remove it, place it into its pocket or wrap it in a cloth.
- Apply to the required area.

HOT WATER:

- Immerse the gel pack in boiling water for a maximum of 5 minutes.
- Remove it, wipe it dry, place it into its pocket or wrap it in a cloth.
- Apply to the required area.

MICROWAVE OVEN:

- Place the pack in the microwave for a maximum of 1 minute at 650 Watts.
- Remove and shake it gently to distribute the heat uniformly.
- Place the pack in the specific pocket or cloth and apply to the required area.

CONTENTS

This is a reusable hot and cold pack which is heated or cooled by an external source. Completely non-toxic and safe for use on the skin.

 ©2021 Dispotech srl
Via al piano, 29
23020 Gordona (SO)
Internet: dispotech.com
Email: info@dispotech

 United Sports Brands
Janssen-Pharmaceuticalaan
Post Box 4
2440 Geel, Belgium
For Info: www.McDavid.eu

 CH Imported by:
United Sports Brands
Switzerland GmbH
Westbahnhofstrasse 1
CH-4500 Solothurn, Switzerland
+32 (0) 14 85 47 85

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REUSABLE HOT-COLD GEL PACK

CLEANING

Wash with lukewarm water and wipe dry.

STORAGE

Store in a cool, dry place when not in use.

DISPOSAL

Check the product for excessive wear before each use. Discard the pack if it shows signs of excessive wear and replace it as necessary.

WHEN TO CONSULT A HEALTHCARE PROFESSIONAL

Consult a healthcare professional if experiencing any reaction to the pack, including skin reaction or changes in condition.

NOTICE TO USER AND/OR PATIENT

 ©2021 Dispotech srl
Via al piano, 29
23020 Gordona (SO)
Internet: dispotech.com
Email: info@dispotech

 United Sports Brands
Janssen-Pharmaceuticaal
Post Box 4
2440 Geel, Belgium
For Info: www.McDavid.eu

 CH Imported by:
United Sports Brands
Switzerland GmbH
Westbahnhofstrasse 1
CH-4500 Solothurn, Switzerland
+32 (0) 14 85 47 85

EC	REP
----	-----

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QNET B.V.
QNET CH-REP GmbH,
QNET LTD.